




# Wound Healing Technologies

## extriCARE 3600 Quick Start Guide


### Turning the device on

Locate the black button on the back of the device and press down. The display screen will light up and say "initializing". Once the device has initialized, the display will show the stand-by screen.

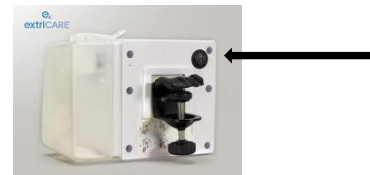

### Setting Therapy to Continuous / Intermittent Mode

The box in the right corner of the display screen shows the Therapy mode setting. A solid blue line indicates continuous mode. A dashed blue line indicates intermittent mode. To change between modes; make sure device is unlocked and suction is off then press and hold the mode button for 2 seconds

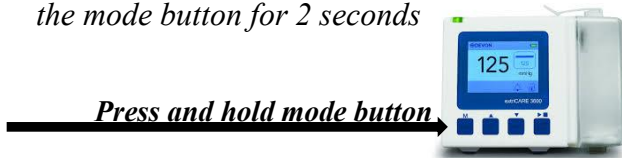

### Starting or Stopping Therapy

To start or stop therapy; make sure device is unlocked then press and hold the START/STOP button for 2 seconds. A blue arrow appears on screen when therapy is actively in progress.



### Unlocking the Device

The lock icon on the display will show a closed lock when the device is locked. To lock / unlock the device; press and hold the both the MODE button and the START / STOP button (for 2 seconds) at the same time.

The device will lock automatically if no button is pressed after 3 minutes. When locked, any button pressed will illuminate the display with the current settings, but the buttons will not take any input, and the backlight to the screen will turn off.



Press and hold both buttons