

## extriCARE®3600 Negative Pressure Wound Therapy Device

In-Service Checklist		
iii Scivic	e encerise	
Facility Name/Location:	Training Date:	
Healthcare Provider (Printed):	Signature:	
Educator (Printed):	Signature:	

	Competency		
3600	extriCare 3600 Device operations		
	Demonstrate turning pump ON and OFF		
	Demonstrate how to read and change the pressure setting		
	Demonstrate how to change between continuous and intermittent modes		
	Demonstrate reading the battery power level		
	Voice understanding of device screen saver feature		
	Demonstrate how to lock/unlock device		
	Demonstrate ability to start/stop therapy		
	Voice understanding of screen display features- Therapy in progress arrow, real-time pressure reading, alarm indication triangle		
	Demonstrate how to install and change canister		
	Voice understanding of canister change frequency		
	Voice need to avoid excess tilt		
	Demonstrate ability to check device for proper function		
	Voice understanding of how to contact Wound Healing Technologies		
	System Troubleshooting		
	Demonstrate how to identify alarms icons		
	Voice understanding of where to locate alarm icon explanations		
	Demonstrate the steps to take to correct alarms		
	Voice understanding of how to contact Wound Healing Technologies		
	Review extriCARE® Clinical Guidelines, Indications and Contraindications		

**Notes:**